


# Van Noort Oesophagostomy Tube Set

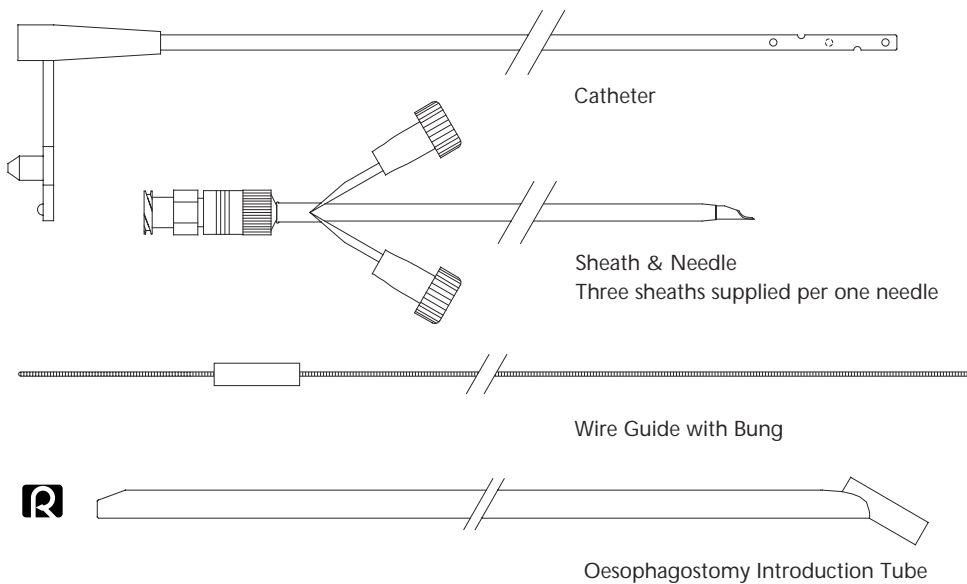

## DESCRIPTION

- An open-end silicone catheter with multiple sideholes used for the administration of enteral solutions.
- Wire guide stylet can be used to stiffen the catheter.
- The introduction tube distal tip is designed for easy palpation. The slot in the tip does not allow the access needle sheath to exit the tip, but does allow the feeding catheter to do so.
- Intended for VETERINARY USE ONLY.

Order No.	Remarks
V-OTCS-1000-VAN-NOORT	Van Noort Oesophagostomy Tube Set, consisting of 10 French 40cm length silicone catheter with wire guide stylet and a 10 gauge Peel-Away® sheath needle.
V-OTC-10-40-VN	Catheter and wire guide only from above set.
V-GTI-8-36.5-IT-VAN-NOORT	Oesophagostomy introduction tube. Not included in set.
V-SNPP-10-5.0-3	10 gauge, 5.0cm long needle with three Peel-Away® sheaths.



1. Introduce tube into midcervical oesophageal area



2. Palpate slot in distal portion of tube



3. Introduce Peel-Away® sheath needle into distal portion of tube



4. Remove needle from sheath



5. Introduce catheter through sheath to the lower third of the oesophagus



6. Peel sheath away and gently remove tube holding catheter in place

## REFERENCE

Dr Ronald van Noort, Diplomate ESVIM, Amsterdam, Netherlands.