


Two-Part Drainage Catheter Sets

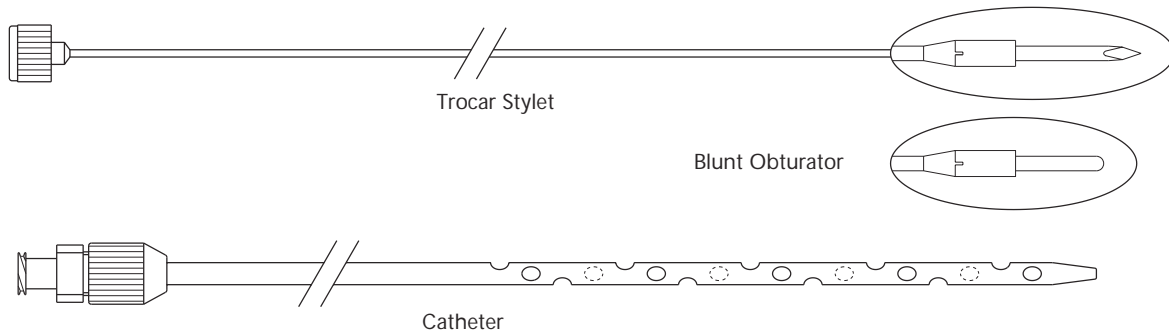

ORDER No.	Remarks
-----------	---------

V-ULT10.2-30-TDC-T45	10.2 French, 30cm long polyurethane thoracic drainage catheter with trocar stylet tip.
----------------------	--

A Multi-Purpose Drainage Set


The trocar stylet allows catheter introduction via a direct puncture. Once the distal end of the stylet and catheter are within the body cavity, the catheter is detached from the stylet and then advanced off the stylet into position within the area requiring drainage.

The blunt obturator allows introduction via direct puncture or to be used to guide the catheter into place following a scalpel incision and blunt dissection.

To attach the catheter to a drainage system, the following attachments are recommended:

- V-POWSC-FLL-MLL-LP one-way plastic stopcock.
- V-FD-100. To prevent tube migration, a fixation disc may be used to fix the drainage catheter at its exit point through the skin. The fixation disc can be sutured in position. A cable tie holds the drain to the disc.

Fenestrated Catheter Wound Drain

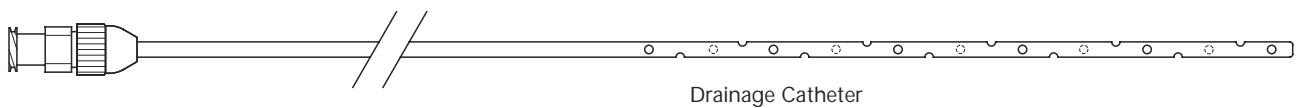

DESCRIPTION

- Polyurethane catheter with multiple fenestrations over the distal two-thirds of the catheter length.
- Suitable for post-orthopaedic wound drainage.
- Catheter material is radiopaque.
- Intended for VETERINARY USE ONLY.

ORDER No.	Remarks
-----------	---------

V-V8.0-NT-25-P-21S-0-WD	8 French, 25cm long fenestrated wound drain.
-------------------------	--